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DURACETAL Instructions

SECTION 1: SYSTEM OVERVIEW

DURACETAL is a pure, highly crystalline acetal copolymer resin possessing high tensile and flexural strength, fatigue resistance, and hardness. These properties, when combined with its very low moisture absorption, make the acetal material ideally suited for milling a variety of removable dental appliances, including partial denture frameworks, provisional crowns and bridges, and bite splints.

SECTION 2: PRODUCT COMPOSITION

Ingrédient	Approx. % weight
Polyoxymethylene Copolymer	99 to 100
Pigments	<1.0

SECTION 3: TECHNICAL DATA

Property	Method	Unit	Value
Density	ASTM D792	g/cm ³	1.41
Tensile Strength (New)	ASTM D638	MPa	61.00
Tensile Modulus	ASTM D638	MPa	2820.00
Flexural Strength (New)	ASTM D790	MPa	90.00
Flexural Modulus	ASTM D790	MPa	2620.00
Moisture Absorption	ASTM D570	wt.%	0.22
Izod Impact Notched	ASTM D256	Ft-lb/in (J/m)	1.00

SECTION 4: INDICATIONS

Removable prosthetic dental appliances

- Partial denture clasp
- Partial denture framework
- Provisional crown & bridge
- Bite splint

SECTION 5: CONTRAINDICATIONS

Contraindications are all applications not included in SECTION 4 above.

SECTION 6: PREPARATION GUIDELINES

General preparation guidelines for all removable appliances should be followed

SECTION 7: DESIGN OF RESTORATION

Partial denture framework (minimum thickness)

- Major connector: 0.7 mm
- Saddle area: 0.5 mm
- Clasp on anterior tooth
- 2.0 mm where clasp joins framework, thinning to 0.7 mm at tip
- Clasp on posterior tooth
- 2.5 mm where clasp joins framework, thinning to 0.7 mm at tip

Temporary Crown & Bridge

- 0.5 mm minimum thickness
- 0.03 mm cement gap

SECTION 8: MILLING

1. Mount the disc according to the CAM-System instructions. Set the enlargement-factor into the unit at a ratio of 1:1.
2. Start the milling unit.
3. Select appropriate milling strategy depending on the appliance to be fabricated.
4. Once the milling operation has completed, remove the disc containing the milled restoration(s).

SECTION 9: POST MILLING

1. Remove the restoration(s) from the disc using a separating disc or suitable acrylic bur.
2. To polish the restoration, load a new polishing wheel or buff with polish suitable for thermoplastic materials and buff all areas of the appliance until all scratches are removed and high shine is achieved.

Notice: All DuraCetal discs are solely for use by or on the order of a dental professional. They are not for use by the public or over-the-counter.

⊗ Milled appliances made from Myerson's discs are not to be re-milled. Re-milling may result in warping since the device will no longer have the support of the body of the puck. Also, the remaining material may not have sufficient retention areas needed for the new device.

SECTION 10: HANDLING & STORAGE

Please refer to SDS

SECTION 11: REGULATORY

ISO 13485.

MANUFACTURED BY
Myerson Company Limited
3 Trinity Avenue, Laventille, Trinidad & Tobago

EUROPEAN REPRESENTATIVE
Obelis s.a.
Bd. Général Wahnis 53, 1030 Brussels, Belgium
phone +32 2 732 59 54
fax +32 2 732 60 03
email: mail@obelis.net

DISTRIBUTED BY
Myerson LLC
5106 North Ravenswood Chicago IL, 60640-2713, USA
toll free phone 800 432 2683

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