



Home



Back



Save



Help

Removables > Frames

Frames

DuraCetal Partial
DuraFlex Denture
VisiClear Denture
RPD Frame

- Add
- Copy
- Delete
- Move up
- Move down

Recycle bin

- No items
- Restore all
- Empty the Recycle Bin

Details

Total: 4 items

Edit RPD Frame

ID 15994_DuraFlex Duraflex

Name

Material

Color

Blockout angle °

Resin gap thickness mm

Relief thickness mm

Liner space mm

Tissue stop diameter mm

Support bar diameter mm

Frame consists of several parts

Basic elements > Materials

Material

- IPS e.max Impulse (Veneer)
- Katana STML
- Katana UTML
- Myerson DuraCetal LT Temp
- Myerson DuraCetal Partial
- Myerson DuraFlex Denture
- Myerson VisiClear Denture
- No material selected
- Zirkon
- TI
- Wax
- PMMA
- CoCr
- Prefab crown
- pritiocrown
- Model Material
- DREVE FotoDentModel Beige
- 3D Systems Visijet
- SnowRock
- Prodways Materials Model320

- + Add
- + Copy
- Delete
- ↑ Move Up
- ↓ Move down

Recycle bin

- No items
- Restore all
- Empty the Recycle Bin

Details

Total: 71 items

Material name

Use default

Color category

Myerson DuraFlex

Logo image
(80x20px size recommended)

Change image Clear image

Not defined

Density

1.00 gr/mm³

Warn if bridge has more than

16 units

Premanufacturing options

Allow to be used with CAD block

Minimum distance to mandrel

2.50 mm

CAD block scaling factor

100 %

Allow to be used with sprue

Locked sprue diameter

Minimum sprue diameter

2.00 mm

Minimum sprue diameter at CAD block

2.00 mm

Minimum manufacturing values (validation)

Write protect validation settings

Always validate at the end of modeling

Minimum wall thickness

0.80 mm

Margin line offset

0.05 mm

Offset angle

85.00 °

Extension offset

0.05 mm

Validation enforcement policy

User can choose to ignore violations

Minimum connector area

1.20 mm²

Use advanced connector validation

[Home](#)[Back](#)[Save](#)[Help](#)**Removables** > **Connectors and Rests****Connectors and Rests**

DuraCetal Connector
 DuraFlex Connector
 VisiClear Connector

-  Add
-  Copy
-  Delete
-  Move up
-  Move down

Recycle bin

-  No items
-  Restore all
-  Empty the Recycle Bin

Details

Total: 3 items

Edit Connectors and Rests

ID	15994_DuraFlex Connector	<input type="button" value="Change ID"/>
Name	<input type="text" value="DuraFlex Connector"/>	
Material	<input type="text" value="Myerson DuraFlex Denture"/>	
Item type	<input type="text" value="Major connector"/>	
Substructure thickness	<input type="text" value="1.20"/>	mm
Wax thickness	<input type="text" value="0.30"/>	mm
Stippling protection	<input type="text" value="0.00"/>	mm